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
I. CONGRATULATIONS

Thank you for choosing Seinbiose to accompany you on your journey through motherhood! At Seinbiose we know that breast milk is the best natural food for your baby's healthy development and growth. Breastfeeding provides your baby with many natural benefits and allows you to enjoy intimate and tender moments together. As your trusted partner, SeinBiose helps mothers give the best to their babies where circumstances make breastfeeding impossible. Discover the ease and benefits of Seinbiose for you and your baby.

II. PRODUCT DESCRIPTION AND FEATURES

- Manual breast pump featuring an easy-to-use pump handle.
- Adjustable 2 level suction control.
- Natural breastfeeding experience.
- Can correct inverted nipples.
- Complete bottle set for easy and convenient nursing.
- Unique air-liquid separation system with anti-backflow.
- Compact and light weight for maximum portability.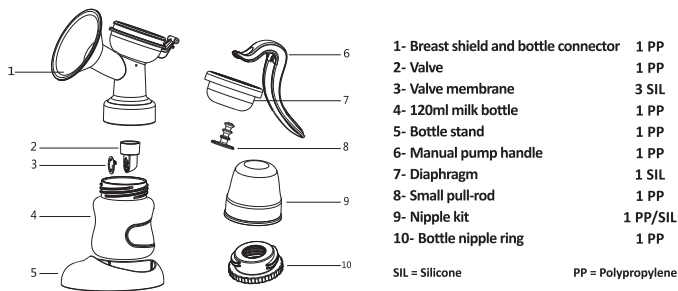

III. IMPORTANT SAFETY INSTRUCTIONS

Please read through the safety instructions carefully before using your Seinbiose breast pump for the first time. Keep these safety instructions for future reference.

- Make sure that all parts are correctly assembled before use.
- Before use, clean all parts in direct contact with the breast milk.
- This device is for personal use. Use by multiple persons may present a health risk.
- Keep the breast pump out of the reach of babies and infants.
- Keep the breast pump away from heat sources and hot surfaces.
- Do not use the breast pump in the bath or shower.
- Do not use the breast pump when sleepy or drowsy.
- Avoid exposing the breast pump and its parts to direct sunlight.
- Do not insert any foreign object into any of the openings of the breast pump and its accessories.


- Do not dry or sterilise the breast pump or its parts in the microwave.
- Discontinue using your SeinBiose breast pump if you experience discomfort or pain during the pumping process and consult your healthcare professional.
- Do not operate the manual breast pump if it is physically damaged or abused.

IV. PREPARING THE PUMP

Ensure that all parts of the breast pump kit that are in direct contact with breast milk are cleaned or sterilised before each use: breast shield and bottle connector (1), valve membrane (3), valve (2), milk bottle (4), nipple kit (9) and bottle nipple ring (10).

(1) Attaching the pull rod

Insert the small pull rod (8) into the opening in the silicone diaphragm (7).

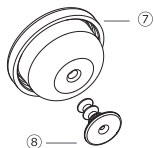(2) Positioning the diaphragm

Position the diaphragm (7) in the opening on the top of the breast shield and bottle connector (1).


(3) Attaching the handle

Insert the small pull rod (8) into the notch on the pump handle (6). Lock the handle into position on the breast shield and bottle connector (1).


(4) Attaching the valve membrane

Place the valve membrane (3) over the valve opening (2). Ensure that the membrane completely seals the valve opening to avoid air leaks.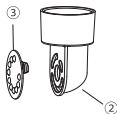

(5) Assembling the pieces

Attach the assembled valve (2) and valve membrane (3) to the bottom of the breast shield and bottle connector (1). Seal tightly. Attach the milk bottle (4) to the bottom of the breast shield and bottle connector. Place the assembled breast pump onto the bottle stand (5).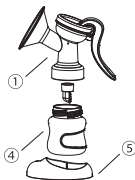


V. USING YOUR SEINBIOSE EASY BREAST PUMP

a. Before use

- Wash your hands before handling the breast pump.
- Verify that the breast pump kit is clean and dry.
- Verify that your breast pump kit is assembled correctly and securely.

b. Using your breast pump

- Find a calm and comfortable place that will allow you to draw your breast milk without being disturbed.
- Place the breast shield onto your breast. Your nipple should be at the centre of the breast shield tunnel to avoid rubbing against the sides. To ensure optimal suction, verify that your breast shield comfortably seals around your breast.
- Use your manual SeinBiose Easy breast pump at a pressure frequency of 60T/ min. Adjust to your comfort level.
- As shown in the illustration, the suction level of your breast pump can be adjusted by attaching the pump handle to the upper or lower position on the small pull rod.
- You can apply two modes during pumping:
 - Stimulation mode: Attach the pump handle to the Upper position on the small pull rod.
 - Expression mode: Attach the pump handle to the Lower position of the small pull rod.

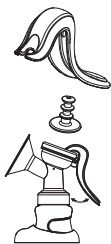Using your breast pump to correct inverted nipples:

- Attach your pump handle to the Lower position of the small pull rod.
- Place the breast shield onto your breast. Your nipple should be at the centre of the breast shield tunnel to avoid rubbing against the sides. To ensure optimal suction, verify that your breast shield comfortably seals around your breast. Press the handle at a steady pace.

c. After use

Once finished, disassemble and clean your breast pump (all parts in direct contact with your breast milk). To clean, follow the cleaning instructions outlined in this user manual.

d. Storing and thawing breast milk

Maternal breast milk provides your baby with the right amount of vitamins, mineral salts, trace elements, sugars, fats and proteins in order to grow and develop in a healthy way. Store and thaw your breast milk appropriately in order to preserve your breast milk's nutritional qualities and to minimise bacterial contamination.

Breast milk can be preserved in the collection cup or bottle provided as part of your breast pump kit, or in freezer-safe containers and bags. Mark the date and hour at which you collected your breast milk on the container before storing it.

SeinBiose recommends the following guidelines for storing and thawing your breast milk :

Breast milk	Room temperature < 26 ° C	Refrigerator + 4 ° C	Freezer - 18 ° C
Freshly expressed milk	4 hours	48 hours	4 months
Thawed in the refrigerator but not heated	Max. 1 hour	24 hours	Do not re-freeze
Thawed and heated	Max. 1 hour	Max. 1 hour	Do not re-freeze
Milk remaining in the feeding bottle	To be consumed within 30 minutes	Throw out	Throw out

Tips for freezing and thawing breast milk :

- Never fill bottles / bags more than $\frac{3}{4}$ full to allow space for possible expansion.
- Label the bottle / bag with the date on which you expressed the milk.
- Do not store breast milk in the refrigerator door. Store it in the lower back area of the refrigerator, where the temperature is coldest.
- Do not heat your breast milk in the microwave. Microwaves may burn your breast milk or alter its nutritional qualities.
- Thaw frozen breast milk in the refrigerator overnight or place it under lukewarm running water (max. 37 degrees). Do not thaw frozen breast milk in boiling water.
- Shake the bottle or bag gently to blend any fat that has separated.
- Use a small cooler with cool-packs to transport your breast milk. Do not exceed a maximal transportation duration of 1 hour.
- For premature or hospitalised babies, consult the healthcare professional accompanying your baby's development or your local milk bank for milk preservation and storage advice.

VI. HOW TO CLEAN YOUR BREAST PUMP

Before using your Seinbiose Easy breast pump, disassemble and wash all parts of the breast pump kit that are in direct contact with the breast milk. Completely air-dry all parts prior to next use.

General cleaning instructions :

- Once finished, disassemble and clean your breast pump (all parts in direct contact with your breast milk). To clean, rinse the pieces of your breast pump with cold water, then wash with warm soapy water using a soft bristle brush. Rinse profusely with clean water and leave to air dry on a clean surface. Once dry, reassemble the pieces for subsequent use.
- Wash the handle after each use. Leave to air-dry completely prior to use.
- All parts of your breast pump can be washed in the dishwasher. Apply a complete washing cycle at 65 degrees Celsius.
- All parts of your breast pump can be washed by hand using warm soap water and a soft-bristle brush.
- Do not use solvents or abrasives.
- Rinse with clean water.

IMPORTANT : Prior to use, visually inspect the individual components of the breast pump kit for cracks, chips, tears, discoloration or general damage and deterioration. Damage to the various components may affect the efficiency of your SeinBiose Easy breast pump. Discontinue use until all parts have been replaced.

VII. TROUBLESHOOTING

If there is no or little suction:

- Verify that all connections are properly attached at the connection points.
- Verify that the membrane is flat and covers the opening of the valve.
- Ensure that the breast shield completely seals around your breast to prevent air leaks.

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